



检查项Action		
1	吸入麻醉剂量是否充足。Inhalant anesthetic level (i.e. Isoflurane) sufficient for the duration of the procedure	<input type="checkbox"/>
2	麻醉罐是否与接管连接紧密，保证气体不能泄露。气体量充足。推荐使用满气体。The carrier gas tank(s) must fit snugly onto the regulator yoke with no leaks and should have sufficient levels for the duration of the procedure.(A back-up "FULL" tank is highly recommended)	<input type="checkbox"/>
3	检查呼吸管是否通畅。Inspect breathing tubing/circuits – free of cracks and holes	<input type="checkbox"/>
4	检查呼吸袋（如适用）Inspect breathing bags (if used) - intact and free from cracks	<input type="checkbox"/>
5	检查回收系统。Inspect scavenge system:Tubing should be free of cracks and holes.	<input type="checkbox"/>
回收Passive System:		
	罐上的排气口是畅通无阻的Exhaust holes on the canister are unobstructed	<input type="checkbox"/>
	罐重已记录Canister weight has been documented	<input type="checkbox"/>
	安全柜运行中Biosafety cabinet/fume hood is turned on and ready	<input type="checkbox"/>
供气系统Active System		
	确保系统正常工作且流速正常Ensure that the vacuum is turned on and working at an appropriate flow	<input type="checkbox"/>